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Extendable RFM bed rail

GB

Read all of the instructions carefully before you assemble, adjust or first use the extendable RFM bed rail.

Intended purpose

The “extendable RFM bed rail” is intended for people who require support moving from a lying to a sitting position. The “extendable RFM bed rail” can be used as a support when getting into or out of bed. It also prevents people from falling out of bed.

Assembly

1. Remove the bed rail from the box and the plastic bag. Check whether it shows any transport damage or defects to the material.

2. Open up the bed rail and place it with the U-shaped section of the frame on the slatted frame of the bed.

3. Open the strap and adjust the width of the frame using the push-button adjustment, then fix the frame in place by attaching the strap to the slatted bed

frame.

4. Fold the side section upwards in a vertical position and fix this in place with the clamping screw.

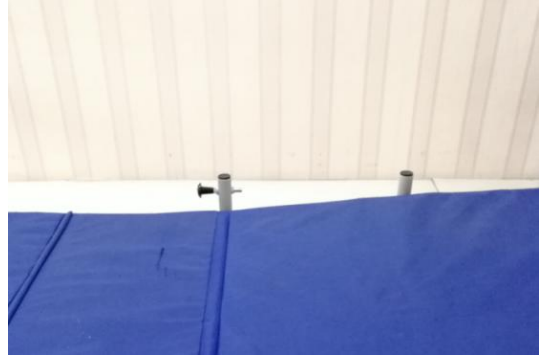
5. The width of this can be altered/extended with the aid of the push-buttons on the side section. It is thus also possible to use the frame on a shorter bed or child's bed as well as on a normal bed. Depending on the person's height, the attachment at the head

end and in the centre of the mattress can be adjusted individually.

6. To drop the rail down, pull the clamping screw out to the side to release the fixing.

Safety information

6. Make sure the "extendable RFM bed rail" is fitted firmly before every use!
7. Do not pull yourself up on the rail from outside and do not put your full body weight on it. The maximum load for the side section is 70 kg, the maximum load of the lower frame under the mattress is 120 kg!
8. To drop down and tighten the side section, slowly loosen the clamping screw in order to avoid injury.
9. The bed rail is not suitable for other uses and children should not hang from it!
10. Do not use a defective extendable RFM bed rail!

Cleaning/multiple use

You should regularly clean and disinfect the extendable RFM bed rail if it is intended for multiple use. Clean it with mild soapy water, a dry cloth and a conventional disinfectant. Do not use any abrasive or corrosive cleaning agents!

Maintenance

The RFM bed rail is maintenance-free. If you identify any damage or changes to the material, please contact your retailer.

Storage

When you are not using the extendable RFM bed rail, store it in the box in a dry place.

Parts supplied

Extendable RFM bed rail
Instructions

Technical data/dimensions and weight

Material: Painted steel
Weight: 3.5 kg
Depth of lower frame: approx. 79 - 89 cm

Height: 50 cm

Height folded: 16 cm

Length (side section): approx. 55 – 72 cm

Diameter of the tubes: 23 mm (outside), 18.6 mm (inside)

Maximum load-bearing capacity: 120kg (frame), 70 kg (side section)

Warranty

The warranty is for 24 months and relates to all product defects that are demonstrably due to manufacturing or material faults. The warranty does not include: Damage that occurs due to natural wear or intent. The warranty also does not include damage that occurs due to incorrect cleaning and care agents, or due to negligent and improper operation or use.

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